


HAIRCARE RESTRUCTURING HAIR BOOSTER

by REVITACARE

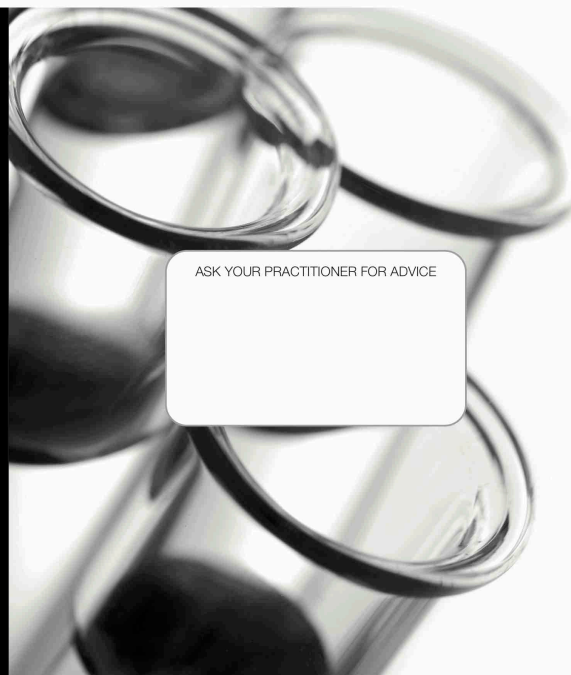

ASK YOUR PRACTITIONER FOR ADVICE

REVITACARE SARL is a French laboratory, founded in 2003, specialized in research, design, manufacture of medical devices based on hyaluronic acid, dedicated to aesthetic medicine and cosmetology to prevent, correct the skin aging effects, and treat hair problems.


HAIRCARE complies with the essential requirements of the Council Directive 93/42/EEC concerning medical devices and its amendments.

QUALITY • SECURITY • EFFICIENCY

- REVITACARE SARL: EN ISO 13485 certified.
- Class III Medical Devices: CE 0499
- HA Quality: biotechnological origin.
- Microbiological safety.
- Biocompatible in compliance with ISO 10993 international standards:
Non cytotoxic - Non irritant - Non sensitizing - Non mutagenic.
- Proved efficiency by clinical data.
- Manufacturing process according to Good Manufacturing Practices (GMP).

LABORATOIRE
REVITACARE
FRANCE

Heliparc - 6 Rue Saint Simon - Parc d'Activités du Vert Galant - 95310 Saint Ouen l'Aumône - FRANCE
Tél: +33 (0) 1 30 37 31 45 - Fax: +33 (0) 1 30 37 88 93 - Email: contact@revitacare.fr
www.revitacare.net

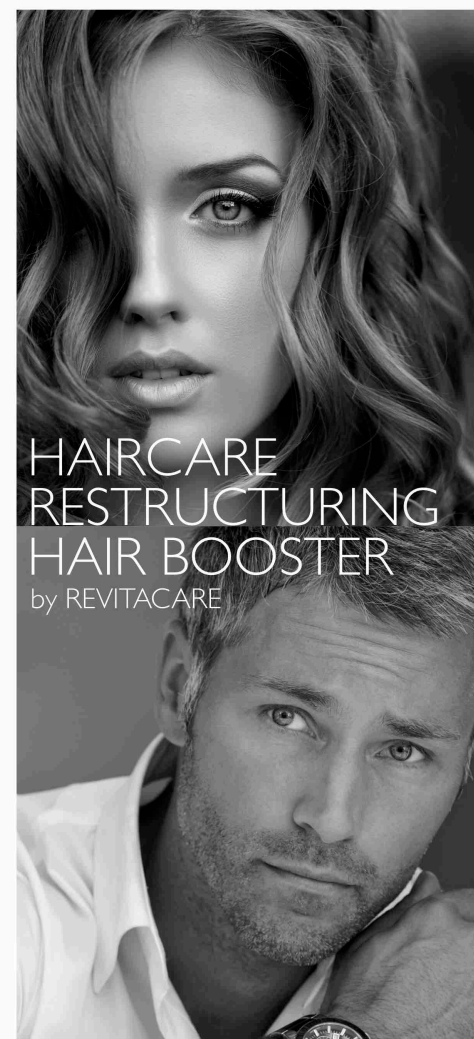

HAIRCARE RESTRUCTURING HAIR BOOSTER

by REVITACARE


LABORATOIRE
REVITACARE
FRANCE

www.revitacare.net


MADE IN FRANCE

REF: AMPR04/03-14-12


HAIRCARE decreases hair loss whatever its origin*.

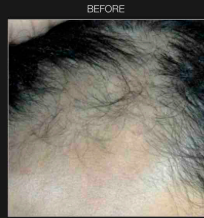


CASE 1 : female patient
After 3 weeks of treatment hair growth is visible.

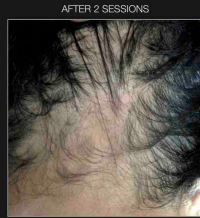


HAIRCARE data source

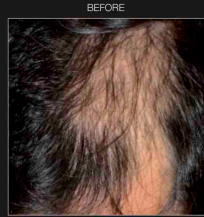


CASE 2 : male patient
After 2 months of treatment hair loss is slowed down and hair regrowth stimulated.

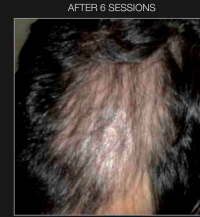

HAIRCARE data source

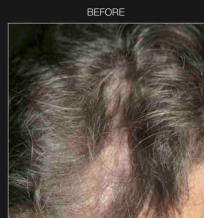


CASE 3 : male patient
After 2 months hair regrowth is accelerated and quality of hair improved.

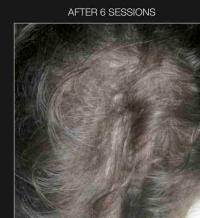

HAIRCARE data source

* Visible results depend on life cycle of hair.

HAIRCARE contributes to the effectiveness of hair implantation by preparing the scalp before surgery, and after intervention, anchoring the new hair into the scalp.

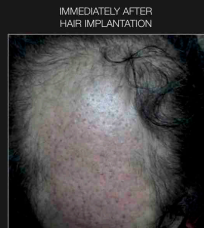

CASE 4 : male patient



HAIRCARE data source

Protocol: 3 HAIRCARE sessions one month before hair implantation, then 15 days after surgery, 1 HAIRCARE session per week during 6 weeks.

STRESS, EXTERNAL AGGRESSIONS (sun, cold, pollution...), UNHEALTHY LIFESTYLE, HORMONAL IMBALANCES...

So many factors which weaken hair that become breakable and dull. Hair loss is accelerated and their growth is slowed down.

INDICATIONS

HAIRCARE is the specific solution for the treatment and attenuation of the symptoms of various scalp and hair problems for both men and women:

- hair loss of whatever the origin,
- dehydration of the scalp and presence of dandruff,
- hair breakage.

SPECIFIC & INNOVATIVE FORMULA

HAIRCARE combines quality of Hyaluronic Acid with its Restructuring Hair Complex¹ to acts on various scalp and hair problems.

RESULTS

Due to its specific and local action:

- hair loss is slowed down,
- hair is restructured,
- scalp is deeply hydrated,
- dandruff is durably eliminated,
- hair regains softness and radiance.

Quality of your hair is improved from its root to its tip, hair regains strength, softness and radiance.

EFFICIENCY CLINICALLY DEMONSTRATED

PATIENTS SATISFACTION

satisfied or highly satisfied by HAIRCARE results*

98% regarding hair loss decrease,

77% regarding scalp rehydration,

81% regarding scalp cleansing,

99% regarding hair texture.

* Multicentric clinical register on 122 patients - Satisfaction level upon protocol completion (6 sessions at intervals of 6 to 10 days)

PROTOCOL

6 sessions at interval of 6 to 10 days are recommended. Further to this protocol, practitioner decides on the frequency of the background therapy, according to severity of the case.

For any questions about contraindications, adverse events, recommendations and precautions for use, please ask your practitioner.



1 - Buffered protective medium: Zinc Gluconate, Arginine, Cysteine, Glutamine, Glycine, Ornithine, Biotin, Calcium Pantothenate, Cyanocobalamin, Folic Acid, Nicotinamide, Pyridoxine.

HAIRCARE is a medical device class III (CE 0499), indicated in the treatment and attenuation of the symptoms of various scalp and hair problems: hair loss of whatever origin, dehydration of the scalp, presence of dandruff and hair breakage, manufactured by REVITACARE SARL. For medical use. Read carefully HAIRCARE leaflet and ask your practitioner for advice.