

SKIN
LIGHTENING
AND
REJUVENATION

CYTOCARE S Line

CYTOCARE
HYALURONIC ACID 18 mg + REJUVENATING COMPLEX
S Line

LABORATOIRE
REVITACARE
FRANCE

1 x 3 ml
MADE IN FRANCE



CYTOCARE
HYALURONIC ACID 18 mg + REJUVENATING COMPLEX
S Line

1 x 3 ml
MADE IN FRANCE

LABORATOIRE
REVITACARE
FRANCE

LABORATOIRE
REVITACARE
FRANCE

MADE IN FRANCE

CYTOCARE

S Line

SKIN LIGHTENING & REJUVENATION



HYDRATING & ANTIOXIDANT
SPECTACULAR LIGHTENING EFFECT

First skin booster sterilized by aseptic filtration presented in a pre-filled syringe to preserve the integrity of its whole components.

- FOR MEN AND WOMEN
- FOR EVERY SKIN TYPE
- FOR EVERY AGE, JUNIOR TO MATURE SKIN



CYTOCARE S Line

Its powerful antioxidant and hydrating formula gives the skin an immediate lightening effect, restoring its physiological balance and qualities. The result is harmonious and natural.


CYTOCARE S Line

revitalizes, hydrates skin of face, neck and back of the hands. Particularly recommended for the rejuvenation of sensitive areas such as peri-oral and peri-orbital areas.

More than an anti-aging product, CYTOCARE S Line has in-depth and on-surface synergic action for full & perfect skin revitalization.

EXCLUSIVE FORMULA

HYALURONIC ACID non-cross-linked + REJUVENATING COMPLEX CT50*

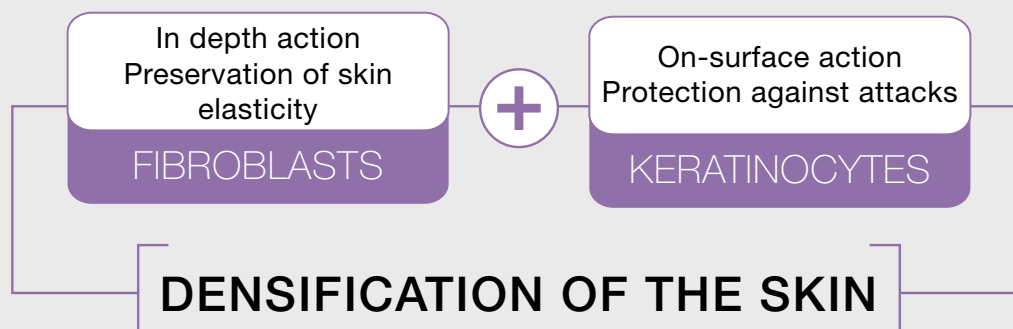


* Ammonium Molybdate, Ammonium Metavanadate, Calcium Chloride, Iron Sulfate, Potassium Chloride, Copper Sulfate, Magnesium Chloride, Manganese Sulfate, Sodium Acetate, Sodium Hydrogen Carbonate, Sodium Chloride, Sodium Hydrogen Phosphate, Sodium Metasilicate, Sodium Selenite, Tin Chloride, Zinc Sulfate, Alanine, Arginine, Asparagine, Aspartic Acid, Cysteine, Glutamine, Glutamic Acid, Glycine, Histidine, Isoleucine, Leucine, Lysine, Methionine, Phenylalanine, Proline, Serine, Threonine, Tryptophan, Tyrosine, Valine, Adenine, Biotin, Calcium Pantothenate, Coline Chloride, Folic Acid, Inositol, Nicotinamide, Pyridoxine, Riboflavin, Thiamine, Cyanocobalamin, Deoxythymidine, Glucose, Putrescine, Sodium Pyruvate, Lipoic Acid.

OPTIMIZATION

CYTOCARE S Line actions thanks to its exclusive formula**

OPTIMIZATION OF ANTIOXIDANT- PROTECTIVE POWER OF HYALURONIC ACID BY REJUVENATING COMPLEX

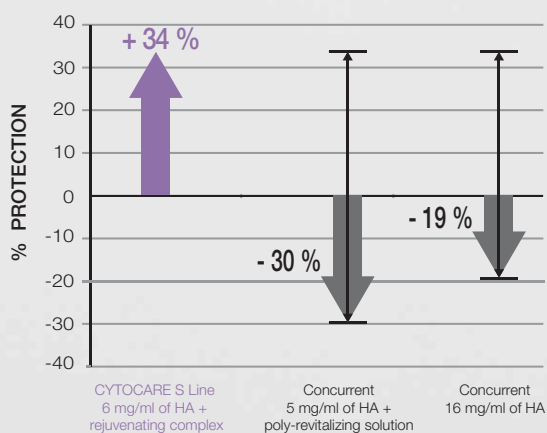


➔ Best antioxidant-protective power *in vivo*.

TABLE 1

Under the experimental conditions of this study, only CYTOCARE S Line has efficiently protected keratinocytes against irradiation, and then is the only one to have antioxidant-protective action.

OPTIMIZATION OF HYDRATING POWER OF HYALURONIC ACID BY REJUVENATING COMPLEX

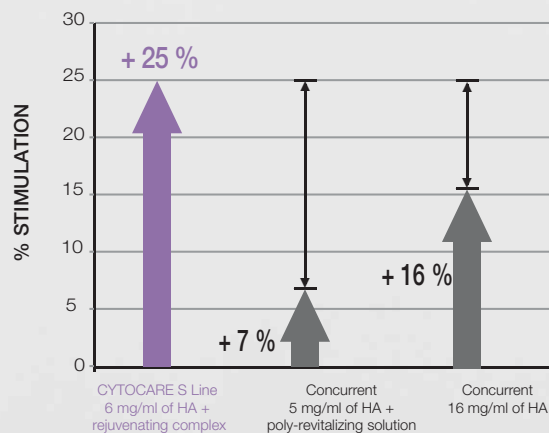

➔ Best hydrating capacity *in vivo*.

TABLE 2

Under the experimental conditions of this study, only CYTOCARE S Line has significantly stimulated sulfated GAGs neosynthesis, which means greater hydrating action.

** *In vitro* studies performed by a French independent laboratory

PRACTITIONERS SATISFACTION

satisfied or highly satisfied by CYTOCARE S Line results*

97%

regarding hydration

97%

regarding improvement of
skin radiance

PATIENTS SATISFACTION

satisfied or highly satisfied by CYTOCARE S Line results*

99%

regarding hydration

98%

regarding improvement of
skin radiance

TOLERANCE

CYTOCARE S Line is very well tolerated.

During the clinical register, bleeding and pain which are bound to injection and not to the product were the most frequent undesirable effects noticed.**


** Multicentric clinical register on 108 patients

PACKAGING

- Sterile single use pharmaceutical grade glass syringe
- Exclusive & secure ergonomic finger grip for comfort and accuracy during injection
- Blister designed to facilitate needles & syringe prehension
- Tyvek reminds easy instruction for use & supports traceability labels

VISIBLE RESULTS

BEFORE

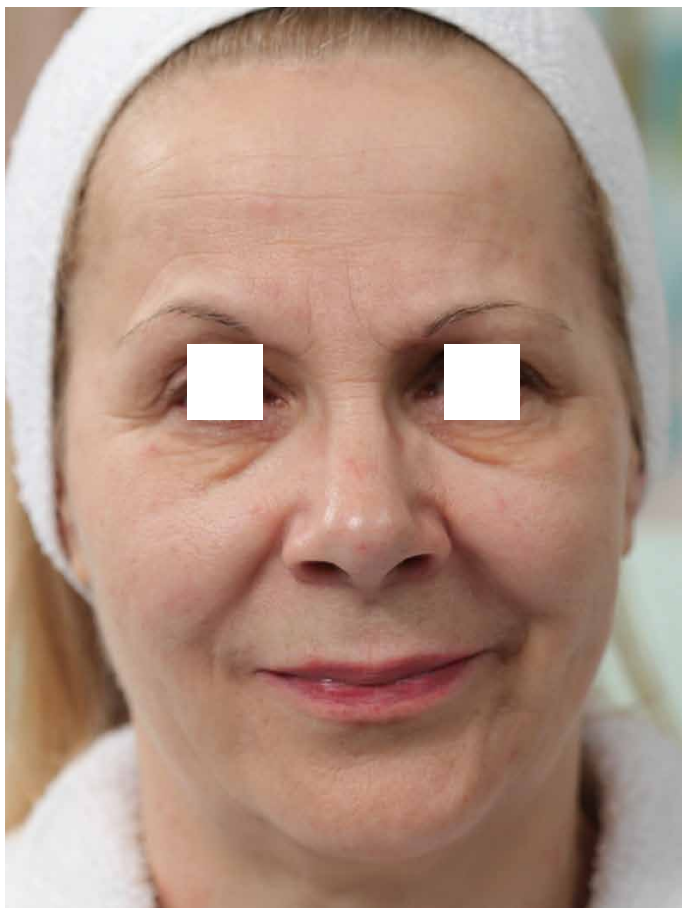

AFTER





TECHNIQUES

- Fan injection
- Cross-hatching
- Retrotracing
- Point-by-point

PROTOCOL

- Injection into the medium or superficial dermis.
- Treatment areas: face, neck and back of the hands.
- Needle: 30G needle supplied with CYTOCARE S Line.
- Protocol: - 1 session for filling fine line; improvement of skin lightening effect.
- For long-lasting result, 3 sessions with 3 weeks intervals for intense skin revitalization.

The practitioner decides on touch-up treatment depending on the maturity of the skin.

CHARACTERISTICS

- Box: 1 x 3 ml syringe + 2 x 30G needles
- Sterilization: aseptic filtration
- Ready to use
- Conservation condition: 2°C to 30°C
- Pyrogen free
- Close to neutral pH
- Iso-osmolar solution

CYTOCARE S Line is a medical device class III (CE 0499) manufactured by REVITACARE SARL, indicated for the prevention and attenuation of wrinkles and fine lines, and for the hydration of the epidermis of face, neck and back of the hands and particularly recommended for the rejuvenation of sensitive areas such as peri-oral and peri-orbital areas. For medical use. Read carefully CYTOCARE S Line leaflet before using the product.

CYTOCARE S Line must be injected in an aseptic manner, by duly practitioner, with sterile and single use equipment supplied by REVITACARE SARL. Before use the product, check the expiry date on the syringe labelling, the integrity of the primary packaging for the syringe and needles and the appearance of the solution. Store between 2°C and 30°C, away from the light in its original secondary packaging. Do not use if package is damaged. Once the CYTOCARE S Line syringe is open, use it immediately. Discard the syringe, needles and the remaining product after use, in accordance with the procedures in force in the establishment. Single use solution: do not re-sterilise and do not re-use. Risks linked to the re-use of the device are infection, inflammation, oedema, cross-contamination, pyrogenicity. CYTOCARE S Line is a medical device class III non-refundable. The practitioner must inform the patient that there are potential adverse effects linked to the injection of our device such as pain, tingling, redness, slight local inflammation, oedema and small haematomas.



REVITACARE SARL is a french laboratory, founded in 2003, specialized in research, design, manufacture of medical devices based on hyaluronic acid, dedicated to aesthetic medicine and cosmetology to prevent, correct the skin aging effects, and treat hair problems

CYTOCARE S Line complies with the essential requirements of the Council Directive 93/42/EEC concerning medical devices and its amendments.

QUALITY SECURITY EFFICIENCY

- REVITACARE SARL: EN ISO 13485 certified.
- Class III Medical Devices: CE 0499.
- HA quality: biotechnological origin.
- Microbiological safety.
- Biocompatible in compliance with ISO 10993 international standards: non cytotoxic - non irritant - non sensitizing - non mutagenic.
- Proved efficiency thanks to *in vitro* studies confirmed by clinical data.
- Manufacturing process according to Good Manufacturing Practices (GMP).

DISTRIBUTOR

LABORATOIRE
REVITACARE
FRANCE

Heliparc - 6 Rue Saint Simon - Parc d'Activités du Vert Galant
95310 Saint Ouen l'Aumône - FRANCE

Tel: +33 (0) 1 30 37 31 45 - Fax: +33 (0) 1 30 37 88 93 - Email: contact@revitacare.fr

www.revitacare.net